

# How to fit Hoofers



- Tie up leg and secure comfortably
- Clean both claws thoroughly with water
- Treat the injured claw
- Prepare healthy claw for Hoofers - trim, scrape clean and **thoroughly dry using meths or a hair-dryer/heat gun**



- Check Hoofers fits the prepared healthy claw. Trim again if necessary



- For small claws, shorten heel of Hoofers (use raised line as guide)
- At this point make sure you have all Hoofers components ready



- The two part glue is mixed in the Hoofers:
  1. Firstly pour all liquid into shoe
  2. Then add 3/4 of powder sachet



- Mix the liquid and powder well
- Mixture should resemble a thick liquid
- Add more powder if mixture is too runny



- Fit the Hoofers on to the healthy claw and apply firm hand pressure for at least 30 seconds
- Sole of Hoofers should be parallel with sole of claw



- Neatly remove excess glue
- Take care to keep glue away from space between claws and the coronet



- Keep leg tied up until glue is set hard
- After untying leg, keep animal standing for a few minutes before allowing it to return to the herd
- Drying times differ with ambient temperature