

# Tubbease Hoof Sock

Treatment aid for a variety of equine hoof problems



The Tubbease Hoof Sock is a revolutionary solution for a variety of equine hoof problems including laminitis and abscesses. The simple, breathable design can be used for tubbing or soaking, or in conjunction with hoof dressings and poultices.

Designed and made in New Zealand and developed by Shoof International Ltd, the hoof sock solves the problem many farriers face of poulticing hooves without 'stewing' them. The EVA-moulded hoof sock 'breathes' whilst still protecting and soaking the damaged hoof, and can be quickly and easily applied without the need for dressings and tape.

Tubbease allows the use of a variety of remedies and can also be used over hoof wraps and dressings for additional protection and cleanliness in wet or muddy conditions. The EVA Sole Insert (sold separately) adds an additional layer of comfort for sensitive soles and will relieve pain and extend the life of Tubbease.

EVA mouldable if placed in hot water before use (60°C).

## Features:

- Soft conforming sock
- EVA overmould
- Hook and loop fastener
- Fast and easy application
- Designed and made in New Zealand

## Benefits:

- Allows the use of powder, cream and liquid remedies, while minimising waste
- Lets hooves breathe naturally
- Hooves stay cleaner and drier in damp conditions and retain natural moisture in dry conditions
- Reduces risk of skin chafing
- Optional EVA insert help relieve pain of sensitive soles, and extend the life of Tubbease



## Fitting Guide (mm)



# Tubbease Hoof Sock

Treatment aid for a variety of equine hoof problems

**Please note:** Measurements indicate maximum internal width of Tubbease.



1. After preparing or medicating the hoof, open sock wide and pull Tubbease over hoof using tabs at side.



2. Check Tubbease is fitting hoof correctly and then pull sock up carefully.



3. With sock taut, fit hook and loop strap to lower pastern with hook side facing sock. Ensure one finger space under strap.



4. With hoof on the ground, simply pour in treatment solution as required, or submerge fitted hoof into a bucket of your chosen solution for deeper soaking.

CODE	PRODUCT
221 162	Tubbease - Pink 110mm each
224 363	Tubbease - Green 130mm each
215 332	Tubbease - Red 145mm each
215 336	Tubbease - Blue 160mm each
221 763	Tubbease - Yellow 175mm each

CODE	PRODUCT
222 959	Replacement Strap - Pink (pair)
222 960	Replacement Strap - Green (pair)
222 961	Replacement Strap - Red (pair)
222 962	Replacement Strap - Blue (pair)
222 964	Replacement Strap - Yellow (pair)

CODE	PRODUCT
221 290	Insert - Pink (pair)
224 362	Insert - Green (pair)
220 967	Insert - Red (pair)
220 968	Insert - Blue (pair)
221 765	Insert - Yellow (pair)



**PLEASE NOTE:** Tubbease Hoof Socks are not enduring boots. They are for short term use in confined enclosures by restrained animals. Judgement should be exercised.